# rego X-ray and ultrasound accessories

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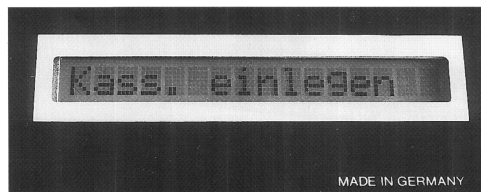
### ● Optimal exposure

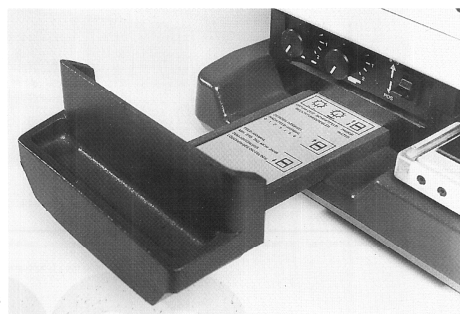
Exposure of X-rays can be adjusted using two separate analogue regulators for the cards and date/time.

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## Safe exposure of X-ray films under daylight conditions makes

labelling of films much easier than complicated methods that label the film in the dark room. The risk of patient confusion is also reduced.

Using the **REGO Identification ID 3 camera**, patient data, the name of the hospital etc. can be inscribed onto the X-ray film in the X-ray cassette. In addition, the LCD Display in the machine automatically records the current **time** and **date** and inscribes **"PA"** in the case of PA views. The whole procedure only takes about 1.5 seconds.

### ● Guidance

The user-friendly guidance system makes using the ID 3 identification camera very simple and safe. **An LCD display and acoustic signals guide the user through the individual steps.** Date and time – exposure in **real time**, protected by a **battery** when the current is switched off. In addition, all the functions are controlled by a **microprocessor** and the user is warned of any possible problems. For instance: *Cassette window jammed > repeat exposure.*

All information can be shown in **German, English, French or Italian.**

### ● Technical Data

Electricity supply: switchable 220–240 volts or 110–120 volts, 50–60 Hz

Dimensions and weight: 280 (h) x 320 (w) x 445 (l) mm, approximately 15 kg

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